

LULLABY

Quick Glance User Guide

Pump must be left on Alternating/Dynamic when patient is on bed unattended



- Ensure Static light is NOT ILLUMINATED when patient left unattended
- Ensure pump is switched to Dynamic

CPR: Ensure CPR is sealed by lining up the white dots



- White = sealed
- Red = open CPR

Power down alarm




Ensure mattress air pipes are connected to power unit correctly



1.

2.

If red/amber alarm light is illuminated, check CPR and air pipes are connected correctly

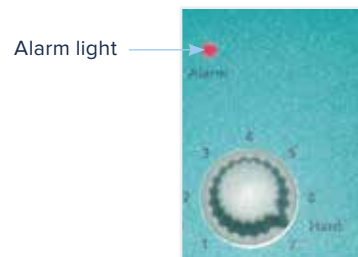

Alarm light

Power down alarm will produce audible and/or red flashing light indicating an interruption in power supply - can be caused by either a mains failure and/or turning off the power unit.

Alarm can be selected using the sliding switch and can be cancelled by either:

- Pressing **RESET** button on power down alarm
- Restoring power supply to the system



Transport mode

Note: Inevitable air loss when disconnecting/connecting pipes can be reduced by increasing pressure before disconnecting pipes

Leaflet Code: LL303

Need Assistance?

Call 01422 233 136 (8:30am - 5pm)

or 0800 037 0234 (out of office hours)

For complete functionality, please refer to the full user guide

LULLABY

Quick Glance User Guide



Siderails

Integral telescopic side rails which can be locked in position at six different heights. Release the side rail by gently sliding the two yellow buttons outwards. While holding the buttons out gently lower/raise the side rail into the desired position and release the buttons.

Ensure that the side rail has locked in position at both ends.

Hinged access point (9003 only)

To position the tubes through the gap in the head/foot end, lift the green latch, position the tubes through the gap and then re-close the latch.

Brakes

There are two linked brake pedals at the foot end of the cot, which control all the castors simultaneously.



Brake

All castors locked.



Free

All four castors are free to move and swivel.



Steer

Locks one castor on the cot so that it cannot swivel.

This allows the cot to be steered when being pushed from the head end of the cot.

Leaflet Code: LL303

Need Assistance?

Call 01422 233 136 (8:30am - 5pm)

or 0800 037 0234 (out of office hours)

For complete functionality, please refer to the full user guide