

# Serenade Cushion 2 & 3

## Quick Glance User Guide

The Serenade 2 & 3 seat cushions are 2 and 3 cell alternating pressure therapy seat cushions. They are suitable for patients up to and including category 4 pressure ulcers and with a patient weight limit of up to 18 stone.

### Set Up

1. If possible, remove the existing seat cushion so that correct seated height is preserved. Fig 1.
2. Position seat cushion on top of the chair with hoses at rear and the foam section at the front.
3. Position the control unit by hanging the hooks over the seat back or place unit on the floor under the chair with the front facing upwards. Ensure the rear of the unit is not obstructed by carpet, rugs etc. It is advisable to place the unit on a firm surface. Fig 2.

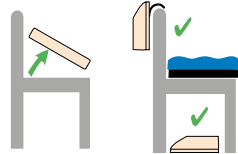


Fig 1

Fig 2

4. Attach the air connector handle to the control unit by depressing the lever on top of the handle and align with the ports on the control box. Ensure air hoses do not kink.
5. Insert the mains cable into the control unit, plug into a mains socket.



Fig 3

### Transport Function

Remove the umbilical cord from the control unit and seal with the transport cap.





### Mains Supply Power Failure



If it is known there is to be a power cut in advance, follow the instructions detailed under the heading 'Transport Function' prior to the power going off.


### In the Event of an Unplanned Power Cut

1. Remove the air connection handle from the control unit and seal with the attached transport cap.
2. Turn off the control unit.

### Operation



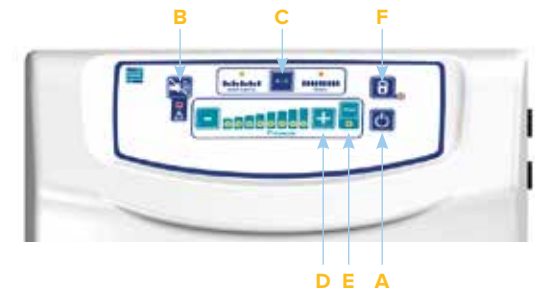


-  Power: Press for 2 secs to switch on/off.
-  Alarm Mute: Mutes the audible alarm for 20 mins. Refer to user guide for detailed instruction.
-  Alternating/Static: Switches between alternating and static. Reverts to alternating after 1 hour.
-  Pressure Adjustment: Press '+' to increase or decrease the pressure setting.

-  Max Inflation: Rapidly inflates mattress to maximum static pressure. Reverts after 20 mins.
-  Lock/Unlock: Locks/unlocks the selected function. Mute remains operational. 'Max' and 'A/S' buttons still revert if selected.



### Troubleshooting

Red alarm light will flash and audible alarm sound.

-  Setup Failure
-  Pressure too Low
-  Pressure too High
-  Alternation Failure
-  Power Failure

Check CPR tag and umbilical cord are attached correctly  
Check all air hoses/cells within mattress. Switch off/on.

Check CPR tag and umbilical cord are attached correctly  
Check all air hoses/cells within mattress. Switch off/on.

Disconnect umbilical cord to reduce pressure.  
Check for twists/kinks in air hoses. Switch off/on

Disconnect umbilical cord to reduce pressure.  
Check for twists/kinks in air hoses. Switch off/on

Check mains cable is plugged in and switched on.  
Check fuse plug and control unit

Leaflet Code: LL306

Need Assistance?

Call 01422 233 136 (8:30am - 5pm)

or 0800 037 0234 (out of office hours)