

Serenade Lite


Quick Glance User Guide




The Serenade Lite Dynamic cushion system is a stand alone dynamic system featuring a a small pump unit suitable for cushion use only. Suitable for users with a patient weight limit of up to 107kg (17 stone).

Set Up

1. If possible, remove the existing seat cushion so that correct seated height is preserved. Fig 1.
2. Position seat cushion on top of the chair with hoses at rear and the foam section at the front. Fig 2.
3. Position the control unit by hanging the hooks over the seat back or place unit on the floor under the chair with the front facing upwards. Ensure the rear of the unit is not obstructed by carpet, rugs etc. It is advisable to place the unit on a firm surface. Fig 2.

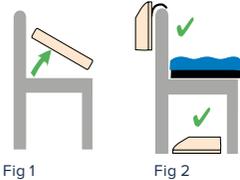

4. Push the air hose connectors onto the control unit, aligning male ends with female ends. Ensure air hoses do not kink. .
5. Insert mains cable into the control unit, plug into a mains socket.

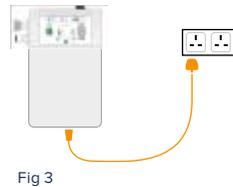

Transport Function

1. Before preparing for transport, switch mode from alternating to static and wait 12 mins for all cells to inflate.
2. Remove all air connectors from the control unit, and connect together. 3. Turn off control unit

Leaflet Code: LL308

Need Assistance?

Call 01422 233 136 (8:30am - 5pm)


or 0800 037 0234 (out of office hours)

Operation

- A. Power:** Pressing the on/off switch turns the control unit on or off. Switch illuminates to indicate the unit is on.
- B. Power Indicator:** The green light illuminates when the system has reached operational pressure.
- C. Warning Indicator:** The red light illuminates when there is a pressure drop in the system. light will remain red until pressure is restored.
- D. Static Mode Indicator:** Orange light illuminates when the cushion is operating in static mode. If static mode indicator is not illuminated the cushion is operating in alternating mode.

- E. Pressure Adjusting Dial:** Adjusts cushion pressure from low to high, in line with patient weight and comfort requirements.
- F. Alternating / Static Switch:** Switches between alternating and static operational modes.

NOTE: This analogue system will not revert back to alternating mode unless it is manually switched back over

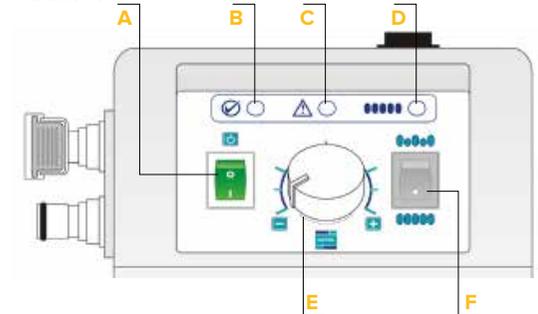

Troubleshooting

Control unit does not operate/power indicator light not illuminated

Check mains cable is correctly attached to wall socket and control unit.

Check mains power is switched on.

Check fuse in both mains plug and control unit. Switch off/on.

Warning Indicator light illuminates

Check air hoses are intact and correctly attached.

Check all air pipes along inside of cushion are correctly attached to air cells.

Check all cells, pipes and hoses for kinks/air leakages. Switch off/on.

Power down

Remove air connectors from control unit and connect together.

Check mains cable is correctly attached to wall socket and control unit.

Check mains power is switched on. Check fuse in both mains plug and control unit. If power is returned, reconnect the air hoses.