

SOLITE^{PRO}

Quick Glance User Guide



The Solite Pro bed is a four section electric profiling bed frame with auto regression, integral bed extension, low height of 39.5cm and a range of safety sides. The bed is suitable for the most dependent patients in a community setting. The lightweight sections make the bed easy to transport, reliable to use and easy to service and clean. The bed has a Safe Working Load of 35.5st.

Installation

Prior to operating the bed for the first time the following simple checks must be performed:


- Ensure the bed and all accessories are at room temperature.
- Ensure the bed has been cleaned and disinfected (see IFU for full instructions).
- Ensure the brakes on the castors at the head end of the bed have been applied.*
- Before locking the castors, ensure they are aligned so that they run parallel to the length of the bed.
- Ensure the mains cable is plugged into an appropriate mains socket.
- The handset may be in a locked or unlocked state, ensure it is set according to the patient requirements
- Using the handset ensure the bed is level, before positioning the bed according to the patient's needs.
- * If the bed will be used in foot down tilt, it is advisable to unlock the foot end castors to prevent them dragging over the floor during the bed's tilting motion, otherwise all four castors should be locked to prevent inadvertent movement of the bed.

Handset Functions

The handset is supplied with a lockout function which enables the carer to disable any of the bed functions if they are deemed unsuitable for the occupant. The lockout function on the handset is to be used at the discretion of the carer.

To lock / unlock a function, insert the handset key provided into the desired function port and turn to the required position (clockwise to lock, anti-clockwise to unlock). When a function has been locked a yellow indicator will appear above that function.

Note: the handset functions are likely to be in the locked state when the bed is first installed.

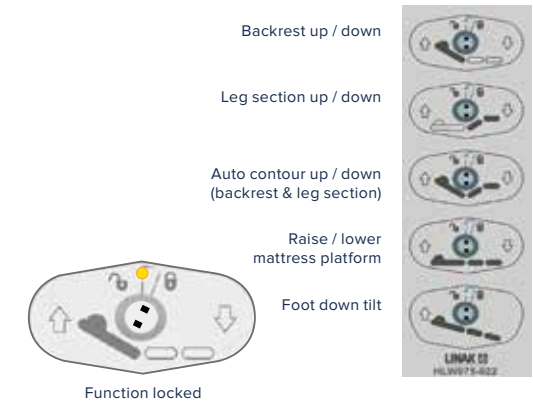

Side Rail Operation

To lower the integral side rails:

1. Lift one end of the top side rail upwards.
2. Depress the release latch at the raised end of the side rail whilst holding the side rail up.
3. Gently lower the side rail until fully down at one end. The release latch can be released after the side rail has begun to lower.
4. Repeat the process at the opposite end of the bed.

To raise the integral side rails:

1. Lift the upper side rail until it is heard to latch into position at the top height at both ends of the bed.



Leaflet Code: LL309

Need Assistance?

Call 01422 233 136 (8:30am - 5pm)

or 0800 037 0234 (out of office hours)



Leg section angle adjustment



Leg section height adjustment

Knee Break / Leg Section

Note: The operation of the leg section is dependent on the position of the ratchets as detailed below.

To set the bed so that the leg section angle adjustment operates:

- Press the leg section button on the handset to fully lower the leg section. The ratchet will automatically default to the angle adjustment setting when the leg section is fully lowered.

To set the bed so that the leg section height adjustment operates:

- Press the leg section button on the handset to raise the leg section to maximum height.

- Taking hold of the bottom of the foot section, slowly lift the section manually so the ratchet engages, stopping when the required angle has been reached.

Note: The ratchet will reset if lifted to its full the leg section height adjustment extent, and will therefore not engage.

The foot section will now be raised as the leg section is driven up / down. The ratchet will automatically reset when the leg section is fully lowered again, defaulting the leg section to the angle adjustment setting.

Brake System

The bed has four braked castors.

- To apply the brakes: Press the brake pedal down on each of the four castors using the foot - Sidhil recommend appropriate footwear is worn to operate the pedal safely.
- To release the brakes: Push the brake release pedal down on each of the four castors (located on top of the castor).

When the bed is in use Sidhil recommend all brakes are applied in normal use – Foot end castors should be unlocked if the tilt function is to be used.



To extend the platform / leg section

- Flatten all sections and lower the bed to its minimum height.
- Ensure the brakes are applied at the head end of the bed.
- Unclip the foot end actuator cable from the bed frame cable clips.
- Remove the 'R' clips, spacers and clevis pins from both sides of the bed frame, towards the foot end of the bed.
- Unlock the foot end castors. Whilst holding the foot end near the locking collars, pull the bed extension out until it reaches its fully extended length.

- Reinsert the 'R' clips, spacers and clevis pins to lock the extension out. If the clevis pin will not fully insert into the frame, then the extension is not fully extended.
- Reconnect the foot end actuator cable to the bed frame cable clips.

Returning the bed to its original state is a reverse of the above.

Leaflet Code: LL309

Need Assistance?

Call 01422 233 136 (8:30am - 5pm)

or 0800 037 0234 (out of office hours)